



DIP & READ PREGNANCY TEST

TEST ANYTIME OF DAY

EASY TO USE

Place test strip in cup with urine for 10 seconds.

FAST RESULTS

In as little as 2 minutes.

FAST RESULTS

(Mon to Fri 8.30am - 5.00pm AEST)
Aus 1800 222 099
NZ 0800 380 218

How the First Response™ pregnancy test works...

The First Response™ Pregnancy Test detects the pregnancy hormone called hCG (human chorionic gonadotropin) in your urine. The amount of pregnancy hormone in your body increases during the first three months of pregnancy. The First Response™ Pregnancy Test is so sensitive that there is usually enough hCG to be detected as early as the first day of your missed period.

Before you begin

PLEASE READ ALL INFORMATION IN THIS LEAFLET BEFORE PERFORMING THE TEST

Do not use after the expiry date stamped on the bottom of the carton.

Store in a dry place below 30°C.

Do not freeze.


Keep out of reach of children.

Single use device. Do not reuse.


For in-vitro diagnostic use.
(Not to be taken internally).


Instructions for use


1. Fill the cup provided with urine.
2. Remove the test strip from the foil wrapper.
3. Hold the Dipping End of the strip in the cup of urine for 10 seconds only. **The urine level should not be higher than the maximum level line printed on the strip.**
4. Remove the strip and place on a flat non-absorbent surface, with the printed side facing up.
5. You should see a pale pink line in the Result Window. This indicates the test is working. The results should be visible in about 2-5 minutes.

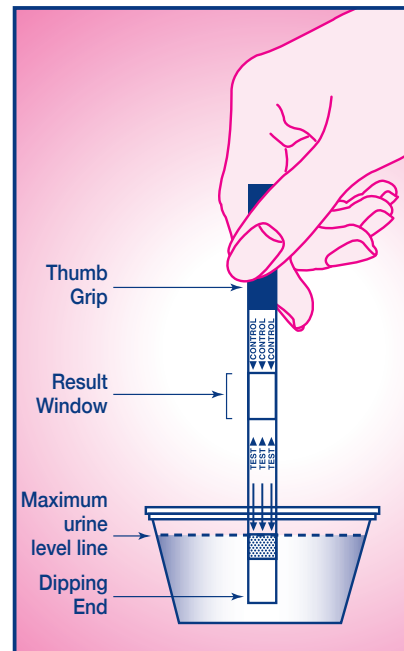

Results

 **Pregnant** - The appearance of two pink lines in the Result Window means you are likely to be pregnant. One line may be lighter than the other, **they do not have to match.**

 **Not Pregnant** - The appearance of one pink line in the Result Window means you are likely to be not pregnant.

 **Invalid** - If no line appears in the Result Window after 5 minutes, the result is invalid and you should repeat the test with another First Response™ Pregnancy Test, ensuring instructions are followed carefully.

Questions & Answers

Q: When can I do the test?

A: You can do the test as early as the first day of your missed period. You can use the First Response™ Pregnancy Test any time of the day. You don't have to use first morning urine.

Q: What do I do if the test result is positive (pregnant)?

A: Remember, this test is not intended to replace your doctor's diagnosis. A positive (pregnant) test result shows that the pregnancy hormone (hCG) was detected. See your doctor to confirm that you are pregnant.

Q: What do I do if the test result is negative (not pregnant)?

A: If your test shows a negative result, no pregnancy hormone (hCG) was detected. You may have miscalculated the length of your cycle or your urine may not have had enough pregnancy hormone (hCG) for the First Response™ Pregnancy Test to give a positive result. If your period does not start in a few days, repeat the test using another First Response™ Pregnancy Test. If you retest and again no hCG is found, and your cycle still has not started, you should contact your doctor.

Q: How long will the test result last?

A: The test result should be read as soon as possible after waiting the 2-5 minutes. A positive result will not disappear. A negative result (1 pink line) will last for 10 minutes. After the 10 minute period, disregard any further changes, it is still a negative result.

Q: Can any medication affect the test?

A: Some medicines containing the pregnancy hormone (hCG) can affect the result of the test. Please consult your healthcare professional if you are taking any medicines. The test should not be affected by hormone therapies containing clomiphene citrate, painkillers, alcohol, antibiotics or the contraceptive pill.

Q: Who can I contact if I have more questions?

A: For further information or advice on using the First Response™ Pregnancy Test, contact: Church & Dwight (Australia) Pty Ltd PO Box 6369, Frenchs Forest, NSW 2086, Australia.

Email: enquiries@churchdwight.com.au

Or telephone us on the

FIRST RESPONSE™ ADVICE LINE

(Monday to Friday 8.30am - 5.00pm AEST)

Australia Free Call: 1800 222 099

New Zealand Free Call: 0800 380 218

www.churchdwight.com.au

www.firstresponsepregnancy.com.au

Important

To increase your chances of a healthy pregnancy for you and your baby, you should consider the following:

- Maintain a well-balanced diet.
- Stop smoking.
- Avoid alcohol consumption.
- The first three months of pregnancy are the most important for your developing baby. You need to see your doctor immediately for proper care and nutrition counselling.

Distributed by:

Church & Dwight (Australia) Pty Ltd
Level 2, 22 Rodborough Road,
Frenchs Forest NSW 2086, Australia.

Manufactured for Church & Dwight Co., Inc.
469 North Harrison Street, Princeton NJ USA

™ FIRST RESPONSE and the
Human Figure Design are trademarks
of Church & Dwight Co., Inc.

U.S. Patent No. 5559041

† Home pregnancy testing market, scan sales data on file.



DOWNLOAD FREE APP First Response™ TRACKER

Helps you track your menstrual cycle,
determine your most fertile days and more.

iTunes

<https://itunes.apple.com/au/app/first-response-tracker/id847529832>

Google Play

<https://play.google.com/store/apps/details?id=com.canadd.firstresponse>