



DIGITAL
PREGNANCY TEST

FIRST TO DETECT TECHNOLOGY
No other test tells you sooner[#]

INSTRUCTIONS AND USE

PLEASE READ ALL THE INFORMATION IN THIS INSERT BEFORE PERFORMING THE TEST.

- Do not use after the expiration date stamped on the bottom of the carton.
- Store in a dry place below 30°C (86°F).
- Do not freeze.
- Keep out of the reach of children.
- Single use device. Do not reuse.
- For in-vitro diagnostic use.
- Not to be taken internally.


If you have any questions please call our advice line.
(Monday to Friday 8.30am - 5.00pm AEST)
AUS: 1800 222 099 NZ: 0800 380 218

WHEN TO USE THE FIRST RESPONSE™ DIGITAL PREGNANCY TEST

The FIRST RESPONSE™ Digital Pregnancy Test is designed to detect hCG (human chorionic gonadotropin) as early as 6 days before your missed period (5 days before the day of the expected period). You can use the FIRST RESPONSE™ test at any time of day. You do not have to use first morning urine. In general, your first-morning urine contains the highest level of pregnancy hormone.

INSTRUCTIONS

THIS DEVICE SELF-ACTIVATES UPON REMOVAL FROM THE WRAPPER. DO NOT OPEN WRAPPER UNTIL YOU ARE READY TO USE.

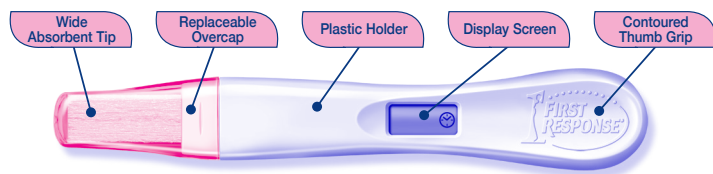


After reading the insert when you are ready to perform the test, remove the test stick from the foil wrapper and remove the **Overcap**. (Upon removal from the wrapper, check the **Display Screen** for the presence of a steady clock symbol ⌚. The device is ready for use. **Do not use the test unless the steady clock symbol is on the Display Screen.**)

STEP 1

EITHER

- Hold **Thumb Grip** so that the **Absorbent Tip** is pointing downward and the **Display Screen** is facing away from your body.
- Place the **Absorbent Tip** in your urine stream for 5 seconds. Be careful not to get the urine on the **Plastic Holder**. During testing, never hold the device with the **Absorbent Tip** pointing upward.

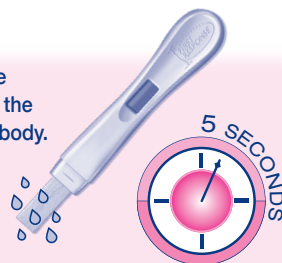

OR

- You may collect your urine in a clean, dry cup and immerse the entire **Absorbent Tip** in the urine for 5 seconds.




READING YOUR RESULTS

READ YOUR RESULTS AT 3 MINUTES

STEP 2

While still holding the stick with the **Absorbent Tip** facing downward, replace the **Overcap** and lay the stick on a flat non-absorbent surface with the **Display Screen** facing up. The clock symbol ⌚ will begin to blink in about 30 seconds after urine is applied to indicate the test is working.

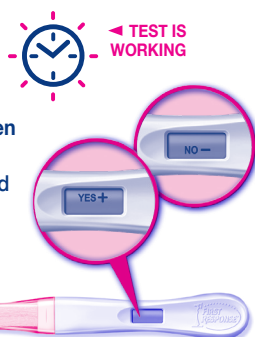

STEP 3

In approximately 3 minutes, an unmistakable 'YES+' or 'NO-' will appear on the **Display Screen**.

HOW TO INTERPRET YOUR TEST RESULT

Pregnant – A positive test result (YES+) means your urine contains hCG, a hormone your body makes during pregnancy. You should consult your doctor to confirm that you are pregnant and for proper prenatal care and nutrition counselling.

Not Pregnant – A negative test (NO-) means that no hCG has been detected and you are probably not pregnant. You may want to repeat the First Response™ Digital Pregnancy Test if a few days pass and you still have not started menstruating. If the result from this later test is still negative, you should see your physician.

[‡] In home pregnancy test kits [#] Home pregnancy testing market, scan sales data on file.

Questions & Answers

Q: How does the test work?

A: The First Response™ Digital Pregnancy Test detects in urine the hormone your body produces during pregnancy; human chorionic gonadotropin (hCG).

Q: When can I test with FIRST RESPONSE™ Digital Pregnancy Test?

A: The First Response™ Digital Pregnancy Test is designed to detect hCG as early as 6 days before your missed period (5 days before the day of the expected period). The amount of pregnancy hormone increases as pregnancy progresses. In laboratory testing with early pregnancy urine samples, First Response gave the following results.

Number of days before the expected period	% of samples from pregnant women giving a positive result
1	>99%
2	>99%
3	96%
4	86%
5	60%

Q: How accurate is the test?

A: First Response™ Digital Pregnancy Test is >99% accurate from the day of the expected period in laboratory studies. However, some pregnant women may not have detectable amounts of pregnancy hormone in their urine on the day they test or may have miscalculated the first day of their period.

Q: What do I do if the result on the Display Screen is 'YES+' (pregnant)?

A: A 'YES+' test result shows that the pregnancy hormone (hCG) was detected, which normally indicates pregnancy. You must see your doctor to confirm that you are pregnant. If you test positive, but think you may not be pregnant, you should check with your doctor.

Q: What do I do if the result on the Display Screen is 'NO-' (not pregnant)?

A: You may not be pregnant, or it may be too early to tell. If you do not get your period within 7 days, you should retest with another First Response™ Digital Pregnancy Test. It is possible that you miscalculated the length of your cycle or your urine may not have had enough pregnancy hormone for the test to register a 'YES+' result. If upon retesting you get another 'NO-' result and your period has still not started, you should call your doctor.

Q: The blinking clock has disappeared followed by a '?' on the Display Screen. What does this mean?

A: A '?' symbol indicates an error has occurred during testing. You should retest with another First Response™ Digital Pregnancy Test, carefully following all directions.

Q: The Display Screen is blank. What is wrong?

A: If the display screen goes blank at any time before yielding a result, an error has occurred. You should retest with another First Response™ Digital Pregnancy Test, carefully following all directions.

Q: How long will my result show on the Display Screen?

A: Your result will be displayed for at least 30 minutes. Read your results within this time frame. Once you have read your result, discard the stick. Note: This device contains a battery

with a removal feature on the reverse side of the Thumb Grip. Be sure to dispose of the battery according to local regulations.

Q: What will happen if the plastic holder gets wet?

A: Some splashing while using the test will not damage the test, but if it becomes very wet it can become damaged. If the test becomes damaged, the Display Screen will either go blank or an error '?' message will appear.

Q: Do any drugs or medications affect the test?

A: Only medications containing the pregnancy hormone (hCG) can affect the result (for example, A.P.L.¹, Pregnyl², Profasi³, and Pergonal³). The test should not be affected by hormone therapies containing Clomiphene citrate (for example Clomid⁴ and Serophene⁵), alcohol, pain killers, antibiotics, or the contraceptive pill.

Q: I removed the device from the wrapper and the clock symbol appeared on the Display Screen; however, I'm not ready to use the test. What should I do?

A: Once removed from the wrapper, the device must be used within 12 hours. Keep the device dry before use.

Q: Who can I contact if I have more questions?

A: For further information or advice on using the First Response™ Digital Pregnancy Test, contact: Church & Dwight (Australia) Pty Ltd
PO Box 6369, Frenchs Forest, NSW 2086, Australia.
Email: enquiries@churchdwight.com.au
Or telephone us on the FIRST RESPONSE ADVICE LINE (Monday to Friday 8.30am - 5.00pm AEST)
Australia Free Call: 1800 222 099
New Zealand Free Call: 0800 380 218
www.churchdwight.com.au
www.firstresponsepregnancy.com.au

Distributed by: Church & Dwight (Australia) Pty Ltd
Level 2, 22 Rodborough Road, Frenchs Forest NSW 2086, Australia.
Manufactured for Church & Dwight Co., Inc.
469 North Harrison Street, Princeton NJ USA
™ FIRST RESPONSE and the Human Figure Design are trademarks of Church & Dwight Co., Inc.

CLINICALLY PROVEN.

- 1 A.P.L. is a registered trademark of Wyeth-Ayerst Laboratories Division of American Home Products Corp.
- 2 Pregnyl is a registered trademark of Organon, Inc.
- 3 Profasi and Pergonal are registered trademarks of Ares Trading SA.
- 4 Clomid is a registered trademark of Merrell Pharmaceutical.
- 5 Serophene is a registered trademark of Serono Laboratories, Inc.

IN0734a

Important

To increase your chances of a healthy pregnancy for you and your baby, you should consider the following:

- Maintain a well-balanced diet.
- Stop smoking.
- Avoid alcohol consumption.
- The first three months of pregnancy are the most important for your developing baby. You need to see your doctor immediately for proper care and nutrition counselling.



DOWNLOAD FREE APP First Response™ TRACKER

Helps you track your menstrual cycle, determine your most fertile days and more.

iTunes

<https://itunes.apple.com/au/app/first-response-tracker/id847529832>

Google Play

<https://play.google.com/store/apps/details?id=com.candd.firstresponse>